


Socket Selection Measurement Guide

TT200 - TRANSTIBIAL PROSTHESIS WITH BUCKLE CLOSURE

The TT200 is our standard buckle closure device for persons measuring 27cm up through 60cm. These sockets will fit limb lengths ranging from 14cm up to 26cm. This socket should be selected for individuals that have measures exceeding the Advanced Closure measurements.

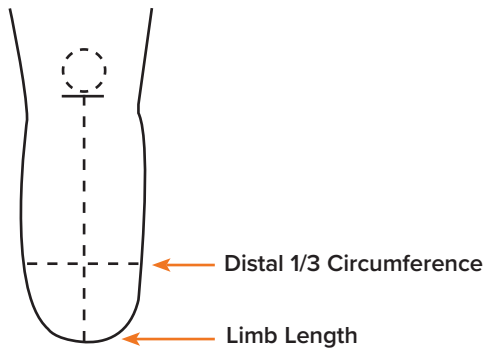

MEASUREMENTS REQUIRED

Record residual limb measures below

Distal 1/3 Circumference _____ cm

Limb Length _____ cm
(Base of patella to end of residual limb)



TT200 (BUCKLE) - SOCKET SELECTION


Distal Circumference	Wearer Weight	Limb Length (base of patella to end)	Socket Size	Select One
27 - 34 cm skin* 29 - 35 cm with liner	260 lb or 118kg	14 - 20 cm	Standard Standard size with small posterior flap	<input type="checkbox"/>
33 - 39 cm skin 35 - 40 cm with liner	260 lb or 118kg	14 - 20 cm	Wide Standard size with spacer and larger posterior flap	<input type="checkbox"/>
38 - 46 cm skin 41 - 50 cm with liner	< 300 lb or 136kg	14 - 20 cm	X-Wide X-wide cup with largest posterior flap	<input type="checkbox"/>
45 - 50 cm skin 46 - 55 cm with liner	< 300 lb or 136 kg	14 - 20 cm	Ultra Wide X-wide cup with 2 of the largest flap, spool adjustment and buckle on right	<input type="checkbox"/>
For persons with longer and larger limbs. Please contact your sales representative to discuss prior to purchase.				
32 - 38 cm skin 34 - 40 cm with liner	< 260 lb or 136 kg	20 - 26 cm	Tall Taller cup with spools and larger posterior flap	<input type="checkbox"/>
32 - 42 cm skin 34 - 44 cm with liner	< 300 lb or 136 kg	20 - 26 cm	TT250 S Uses spool and buckle closure	<input type="checkbox"/>
42 - 58 cm skin 44 - 60 cm with liner	< 300 lb or 136 kg	20 - 26 cm	TT250 W Largest prosthesis offered standard and wide – uses spool and buckle closure	<input type="checkbox"/>

*Persons smaller than this measure will require a Narrow/Pediatric socket. Please refer to the Advanced Closure sizing guide.

Socket Selection Measurement Guide

TT ADVANCED CLOSURE (TTAC) TRANSTIBIAL PROSTHESIS WITH INTERNAL PULLEY MECHANISM

The TT Advanced Closure system is our latest transtibial socket which features a standard adult size for persons 14-26cm in limb length and 26-34cm in circumference while wearing their liner. This line also features a Narrow/ Pediatric socket designed for children or persons weighing under 120 pounds. These sockets will fit limb lengths 14-26cm and circumferences 20-25cm with liner. Custom sockets can also be requested for persons with symes amputation.

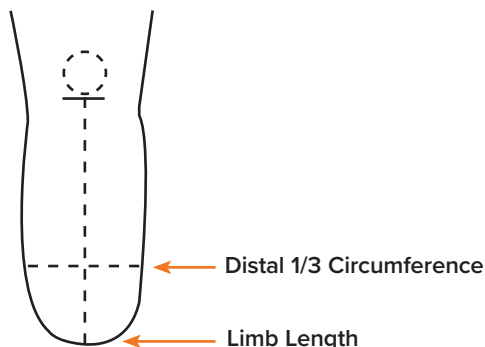

MEASUREMENTS REQUIRED

Record residual limb measures below

Distal 1/3 Circumference _____ cm

Limb Length _____ cm
(Base of patella to end of residual limb)



ADVANCED CLOSURE SYSTEM SIZING - SOCKET SELECTION


Distal Circumference	Wearer Weight	Limb Length (base of patella to end)	Socket Size Transtibial Advanced Closure	Select One
Narrow or Pediatric Socket - Ideal for pediatric patients (children and teens) or adults with narrow limbs.				
18 - 24 cm on the skin or 20 - 26 cm with liner	Must weigh less than 120 pounds (54kg)	12 cm - 15 cm	TTAC - N - S Narrow - Short	<input type="checkbox"/>
		16 cm - 19 cm	TTAC - N - M Narrow - Medium	<input type="checkbox"/>
		20 cm - 24 cm*	TTAC - N - T Narrow - Tall	<input type="checkbox"/>
Standard Adult Socket				
25 - 32 cm on the skin or 27 - 34 cm with liner	Must weigh less than 260 pounds (125kg)	14 cm - 17 cm	TTAC - S - S Standard - Short	<input type="checkbox"/>
		18 cm - 22 cm	TTAC - S - M Standard - Medium	<input type="checkbox"/>
		23 cm - 26 cm*	TTAC - S - T Standard - Tall	<input type="checkbox"/>
Wide Adult Socket				
32 - 38 cm on the skin or 34 - 40 cm with liner	Must weigh less than 260 pounds (125kg)	14 cm - 17 cm	TTAC - W - S Wide - Short	<input type="checkbox"/>
		18 cm - 22 cm	TTAC - W - M Wide - Medium	<input type="checkbox"/>
		23 cm - 26 cm*	TTAC - W - T Wide - Tall	<input type="checkbox"/>

*Longer limb lengths require a custom socket. Please speak with your sales representative.

Socket Selection Measurement Guide

TF300 TRANSFEMORAL SOCKET

The transfemoral socket will fit limb lengths 18-30+ cm and distal circumferences 30 - 60 cm with liner. This system features a buckle closure and can be purchased with either cable and notches or the spool closure.

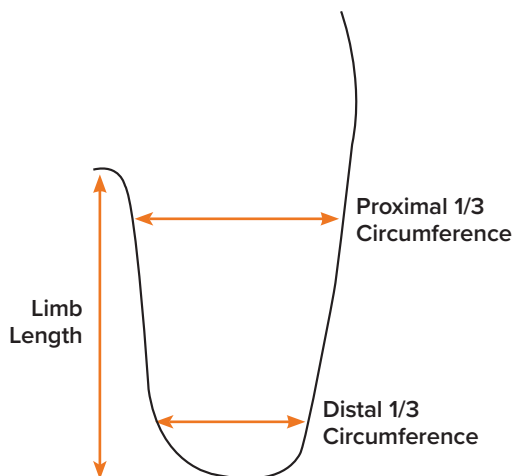

MEASUREMENTS REQUIRED

Record residual limb measures below

Proximal 1/3 Circumference _____ cm

Distal 1/3 Circumference _____ cm

Limb Length* _____ cm
(Adductor origin to end of residual limb)



TF300 - SOCKET SELECTION

Distal Circumference	Proximal Circumference	Patient Weight	Socket Size	Select One	
				Cable with Hooks	Spool System
30 - 44 cm with liner	35 - 50 cm	260 lb or 118 kg	Standard	<input type="checkbox"/>	<input type="checkbox"/>
43 - 60 cm with liner	50 - 75 cm	260 lb or 118 kg	Wide	<input type="checkbox"/>	<input type="checkbox"/>