


Socket Selection Measurement Guide

TT200 - TRANSTIBIAL PROSTHESIS WITH BUCKLE CLOSURE

The TT200 is our standard buckle closure device for persons measuring 27cm up through 60cm. These sockets will fit limb lengths ranging from 14cm up to 26cm. This socket should be selected for individuals that have measures exceeding the Advanced Closure measurements.

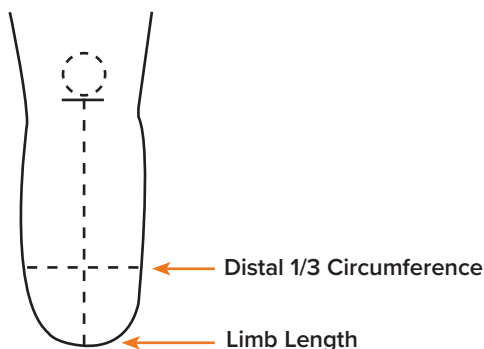

MEASUREMENTS REQUIRED

Record residual limb measures below

Distal 1/3 Circumference _____ cm

Limb Length _____ cm
(Base of patella to end of residual limb)



TT200 (BUCKLE) - SOCKET SELECTION


Distal Circumference	Wearer Weight	Limb Length (base of patella to end)	Socket Size	Select One
27- 34 cm skin* 29 - 36 cm with liner	260 lb or 118kg	14 - 20 cm	Standard Standard size with small posterior flap	<input type="checkbox"/>
27 - 34 cm skin 29 - 36 cm with liner	260 lb or 118kg	20 - 26 cm	Tall Taller cup for long limbs	<input type="checkbox"/>
33 - 39 cm skin 35 - 40 cm with liner	260 lb or 118kg	14 - 20 cm	Wide Standard size with spacer and larger posterior flap	<input type="checkbox"/>
38 - 46 cm skin 41 - 50 cm with liner	300 lb or 136 kg	14 - 20 cm	X-Wide X-wide cup with largest posterior flap	<input type="checkbox"/>
45 - 50 cm skin 46 - 55 cm with liner	300 lb or 136 kg	14 - 20 cm	Ultra Wide X-wide cup with 2 of the largest flap, spool adjustment and buckle on right	<input type="checkbox"/>
45 - 60 cm skin 46 - 62 cm with liner	300 lb or 136 kg	20 - 26 cm	TT250 Largest prosthesis offered – uses spool and buckle closure	<input type="checkbox"/>

*Persons smaller than this measure will require a Narrow/Pediatric socket. Please refer to the Advanced Closure sizing guide.

Socket Selection Measurement Guide

TT ADVANCED CLOSURE (TTAC) TRANSTIBIAL PROSTHESIS WITH INTERNAL PULLEY MECHANISM

The TT Advanced Closure system is our latest transtibial socket which features a standard adult size for persons 14- 26cm in limb length and 26 -34cm in circumference while wearing their liner. This line also features a Narrow/ Pediatric socket designed for children or persons weighing under 120 pounds. These sockets will fit limb lengths 14 -26cm and circumferences 20-25cm with liner. Custom sockets can also be requested for persons with symes amputation.

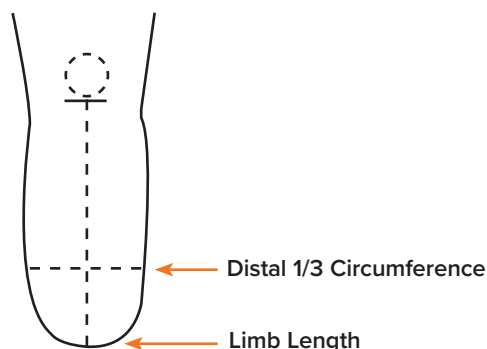

MEASUREMENTS REQUIRED

Record residual limb measures below

Distal 1/3 Circumference _____ cm

Limb Length _____ cm
(Base of patella to end of residual limb)



ADVANCED CLOSURE SYSTEM SIZING - SOCKET SELECTION

Distal Circumference	Wearer Weight	Limb Length (base of patella to end)	Socket Size	Select One
----------------------	---------------	--------------------------------------	-------------	------------

Narrow/Pediatric Socket - Ideal for children, teens and adults with narrow limbs

18 - 24 cm skin 20 - 25cm with liner	Less than 120 lb or 54 kg	14 cm - 18 cm	TTAC - N - M Transtibial Advanced Closure Narrow - Medium	<input type="checkbox"/>
		19 cm - 24 cm*	TTAC - N - T Transtibial Advanced Closure Narrow - Tall	<input type="checkbox"/>

Adult Socket

25-31 cm skin 26 - 34 cm with liner	Less than 260lb or 118 kg	14 cm - 17 cm	TTAC - S - S Transtibial Advanced Closure Standard - Short	<input type="checkbox"/>
		18 cm - 22 cm	TTAC - S - M Transtibial Advanced Closure Standard - Medium	<input type="checkbox"/>
		23 cm - 26 cm*	TTAC - S - T Transtibial Advanced Closure Standard - Tall	<input type="checkbox"/>

*Longer limb lengths require a custom socket. Please speak with your sales representative.